
Microlife NEB 1000

microlife

NEB 1000

IB NEB 1000 EN-PL 1517

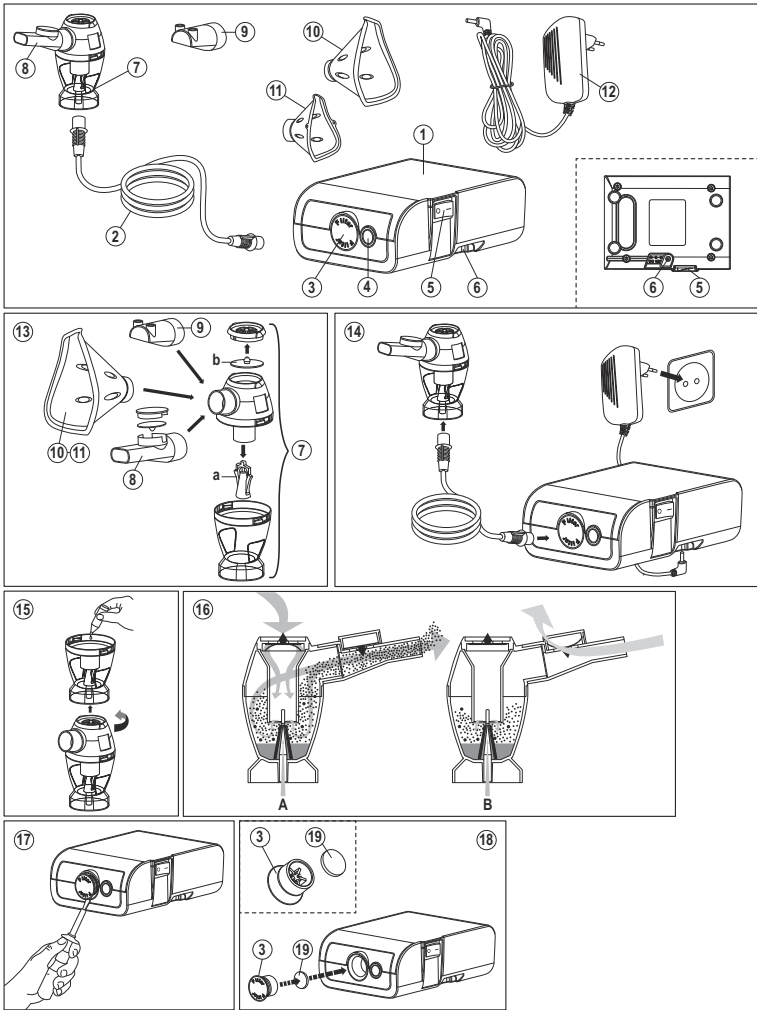Guarantee Card

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Portable Compressor Nebulizer

EN

- 1 Piston compressor
- 2 Air hose
- 3 Air filter compartment
- 4 Air hose connector
- 5 ON/OFF Switch
- 6 AC adapter socket
- 7 Nebuliser «NEB PRO»
 - a: Vapouriser head
 - b: Membrane
- 8 Mouthpiece with valve
- 9 Nose piece
- 10 Adult face mask
- 11 Child face mask
- 12 AC adapter
- 13 Assembling nebuliser kit
- 14 Assembling the device
- 15 Filling the nebuliser
- 16 Breath-enhanced nebulisation
- 17 Opening the air filter compartment
- 18 Replacing air filter
- 19 Air filter

Dear Customer,
This nebuliser is a high-quality device for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses. The operation and usage of the device is very straightforward. All common liquid medication can be used for inhalation therapy. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your Microlife product. If you have any questions, problems or want to order spare parts please contact your local Microlife-Customer Service. Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – Microlife AG!

- Read the instructions carefully before using this device.
- Type BF applied part
- Keep dry

1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device and the AC adapter if you think they are damaged or notice anything unusual.
- Never open this device and the AC adapter.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
 - water and moisture
 - extreme temperatures
 - impact and dropping
 - contamination and dust
 - direct sunlight
 - heat and cold
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Only use the AC adapter model SJ-1220-E (cod. 3A3309). Do not use this device with other AC adapters. Keep the cord away from hot surfaces.
- Never plug in or unplug the AC adapter with wet hands. Never use this device (with the AC adapter inserted) close to water, do not get the device wet or immerse it in any liquids. If by accident it falls into water, pull the plug out of the wall socket immediately before recovering it.
- Never bend the nebuliser over 60°.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.

- Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
- Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Use of this device is not intended as a substitute for a consultation with your doctor.

2. Preparation and Usage of this Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

1. Assemble the nebuliser kit (13). Ensure that all parts are complete.
2. Fill the nebuliser (15) with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
3. Connect the nebuliser (7) with the air hose (2) to the air hose connector (4) of the compressor (1), as shown in figure (14).
4. Connect the AC adapter (12) to the AC adapter socket (6) of the device and plug the power plug of AC adapter into the wall socket.
5. Switch the ON/OFF switch (5) to position «I» to turn on the device and place the mouthpiece (8) in the mouth or fit one of the face masks over mouth and nose.
 - The mouthpiece gives you a better drug delivery to the lungs.
 - Choose between adult (10) or child face mask (11) and make sure that it encloses the mouth and nose area completely.
 - Use all accessories including the nose piece (9) as prescribed by your doctor.
6. Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. **Do not lie down while inhaling.** Stop inhalation if you feel unwell.
7. After completing the inhalation period recommended by your doctor, switch the ON/OFF switch (5) to position «O» to turn off the device. Disconnect the AC adapter (12) from the wall socket and from the device. Disconnect the air hose (2) from the nebuliser (7) and from the air hose connector (4) of the device.
8. Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».

This device was designed for intermittent use of 30 min. On / 30 min. Off. Switch off the device after 30 min. use and wait for another 30 min. before you resume treatment.

3. Nebuliser «NEB PRO» treatment options

This device is equipped with the innovative nebuliser «NEB PRO» (7) which includes the breath-enhanced valve system (16). All common medication for inhalation therapy can be nebulised effectively.

Advanced treatment
This treatment option allows the maximum amount of medication to be delivered thus reaching the respiratory capacity of each patient. The valve system optimally adapts the flow of medication during inhalation (16-A, reducing medication loss during exhalation (16-B). Use the breath-enhanced valve system (synchronized with breathing) to optimize the therapeutic effects of the medication and to get the maximal benefit.

- ▶ Mount the membrane on the valve-holder and use the mouthpiece with expiratory valve (8).
- ▶ Follow the usage steps as described in «Section 2.»

Only using the mouthpiece with expiratory valve (8) enables the synchronized functioning of the nebuliser.

Short treatment
The therapeutic session will be reduced if the membrane is not mounted on the valve-holder.

A synchronized nebulisation is not possible using this treatment option.

4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. The compressor (1) and the air hose (2) should be cleaned with a clean, moist cloth.

- Always wash your hands well before cleaning and disinfecting the accessories.
- Do not expose the compressor to water or heat.
- Replace the air hose for each treatment with a new patient or in case of impurities.

Always unplug the AC adapter from the power supply before cleaning.

- Cleaning with water**
- ▶ Wash all nebuliser components (except air hose) under warm tap water (max. 60 °C) for about 5 minutes adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer.
 - ▶ Rinse thoroughly making sure that all deposits are washed away and leave to dry.

Disinfecting

- ▶ All nebuliser components (except air hose) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer.
- ▶ Disinfectants are usually available at pharmacies.

- Sterilizing with steam**
- ▶ All nebuliser components (except air hose and masks) can be heat steam sterilized up to 121 °C (20 min.) or 134 °C (7 min.). EN554/ISO11134.
 - ▶ The sterilisation packaging must conform to EN868/ISO11607 and be suitable for steam sterilisation.
 - ▶ After sterilization always let all components cool down to ambient temperature before further use.
 - Do not repeat sterilization cycle when components are still warm.

5. Maintenance, Care, and Service

- Order all spare parts from your dealer or pharmacist, or contact Microlife-Service (see forward).
- ▶ We recommend replacing the nebuliser, mouthpiece, and the face masks after 3 months use.
 - ▶ Check the filter continually for cleanliness and replace it if dirty, or after a maximum of 3 months use. Spare filters are provided with the device.
 - ▶ To replace the air filter (19), extract the air filter compartment (3) from the device using a screwdriver (17). Exchange the used air filter with a new filter. Gently place the new filter into the inner part of the compartment (18) and make sure that it is correctly fixed.

6. Malfunctions and Actions to take

- The device cannot be switched on**
- Ensure the AC adapter (12) is correctly plugged to the AC adapter socket (6) of the device and the power plug into the wall socket.
 - Ensure the ON/OFF switch (5) is in the position «I».
- The nebuliser functions poorly or not at all**
- Ensure the air hose (2) is correctly connected at both ends.
 - Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one.
 - Ensure the nebuliser (7) is fully assembled and the vapouriser head (7-a) is placed correctly.
 - Ensure the required medication has been added.

7. Guarantee

This device is covered by a 3 year guarantee from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.

- The guarantee covers only the compressor. The replaceable components like nebuliser, masks, mouthpiece, air hose, and filters are not included.
- Opening or altering the device invalidates the guarantee.
- The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

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