
# Mesh Nebuliser

## NEB 800

microlife

### NEB 800

IB NEB 800 EN-PL 1017

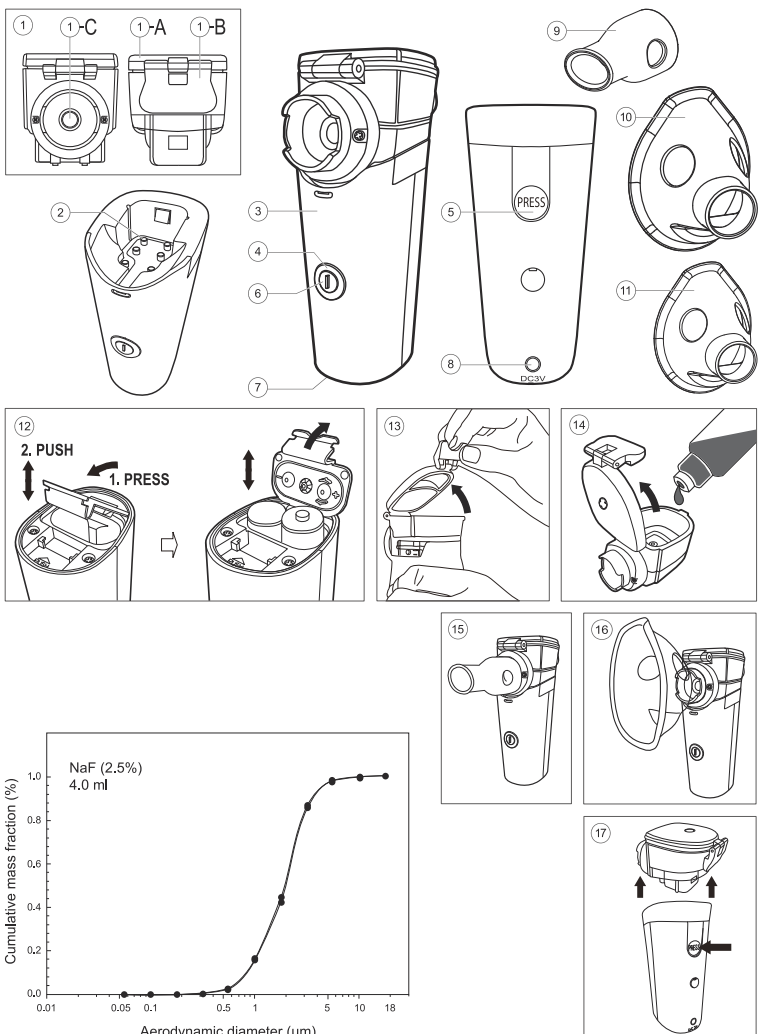#### Guarantee Card

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Obelis s.a.  
Bd Général Wahis 53, 1030 Brussels / Belgium

## Mesh Nebuliser

- ① Nebulising head  
-A: Cover, -B: Lock, -C: Metal mesh
- ② Electrodes
- ③ Main unit
- ④ Indicator light
- ⑤ «PRESS» button
- ⑥ ON/OFF button
- ⑦ Battery Compartment
- ⑧ Mains Adapter Socket
- ⑨ Mouthpiece
- ⑩ Adult face mask
- ⑪ Child face mask
- ⑫ Inserting the batteries
- ⑬ Opening the nebulising head
- ⑭ Filling the nebulising head
- ⑮ Connecting the mouthpiece
- ⑯ Connecting the mask
- ⑰ Removing the nebulising head

Dear Customer,  
This mesh nebuliser is intended for inhalation therapy for asthma, chronic bronchitis, and other respiratory illnesses. Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your Microlife product. If you have any questions, problems or want to order spare parts please contact your local Microlife-Customer Service. Your dealer or pharmacy will be able to give you the address of the Microlife dealer in your country. Alternatively, visit the internet at [www.microlife.com](http://www.microlife.com) where you will find a wealth of invaluable information on our products. Retain instructions in a safe place for future reference. Stay healthy – Microlife AG!

Read the instructions carefully before using this device.

Type BF applied part

### 1. Important Safety Instructions

- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect nebulising.
- Never touch the metal mesh in the nebulising head with your hands or foreign objects.
- This device should only be used with original accessories as shown in these instructions.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Only use the medication prescribed for you by your doctor and follow your doctor's instructions with regard to dosage, duration and frequency of the therapy.
- Do not use following medication: NEBUDOSE hyaluronic, NEBUDOSE hipertonic.
- The liquid for nebulising should be prepared based on 0.9% saline or whatever advised by your doctor. No tap or boiled water can be used. The container used for the preparation of liquid must be sterilized.
- Never use liquids which contain ester, fatty or suspended particles (including diffusion juice of herbs); specifications may vary with the medication type used.
- If the device is not going to be used for a prolonged period the batteries should be removed.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- Protect it from:
  - water and moisture
  - extreme temperatures
  - impact and dropping
  - contamination and dust
  - direct sunlight
  - heat and cold
  - flammable gas, heating units and open fire
  - corrosive liquids
- To avoid any risk of electric shock when using this device please observe the following:
  - Never use the device in a humid environment such as a bathroom nor when the power cord or the main unit has gotten wet.
  - Never plug in or unplug the adapter cable with wet hands.
  - Check that the supply voltage at the power socket is correct, as not to cause overload to the device.
  - Unplug the device from the power supply before cleaning.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Use of this device is not intended as a substitute for a consultation with your doctor.

### 2. Preparation before Use

Inserting the batteries ③  
After you have unpacked your device, first insert the batteries. The battery compartment (⑦) is on the bottom of the device. Insert the batteries (2 x 1.5 V, size AA), thereby observing the indicated polarity.

Using a Mains Adapter  
You can operate this device using the Microlife mains adapter (DC 3V, 1.33A).

Only use the Microlife mains adapter available as an original accessory appropriate for your supply voltage.

Ensure that neither the mains adapter nor the cable are damaged.

1. Plug the adapter cable into the mains adapter socket (⑧) in the main unit (③).
2. Plug the adapter plug into the wall socket.

**Trial run**  
Press the ON/OFF button (⑥) and check if the indicator light (④) lights up green.

**Orange light:** batteries are weak.  
**Red light:** batteries are exhausted and need replacing.

**Red light flashes 3 times:** device turns off automatically, as no medicine is inside.

### 3. Using the Device for the First Time

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

1. Snap the nebuliser lock (①-B) upward to open the nebulising head (③).
2. Fill the medication as per your doctor's instructions into the drug storage area (④). Ensure that you do not exceed the maximum level.
3. Press the nebuliser lock (①-B) downwards; a «click» sound indicates that the nebuliser cover (①-A) is closed correctly.
4. Connect the mouthpiece or a face mask as shown in picture (⑩) and (⑪).
- The mouthpiece gives you a better drug delivery to the lungs.
- Choose between adult (⑩) or child face mask (⑪) and make sure that it encloses the mouth and nose area completely.
5. Start the nebulisation by pressing the ON/OFF button (⑥); the indicator light (④) will light up green.
6. Breathe in and out calmly during the therapy. Sit in a relaxed position with the upper body upright. **Do not lie down while inhaling.** Stop inhalation if you feel unwell.
7. To stop the nebulisation during the therapy, press the ON/OFF button (⑥); otherwise the device will automatically shut down, when the liquid has been fully used.
8. If the Microlife mains adapter is used, unplug the cable from the mains adapter socket (⑧).
9. Remove the mouthpiece (⑩) or mask (⑩ / ⑪).
10. Press the «PRESS» button (⑤) on the rear of the main unit (③) and gently pull the nebulising head (①) out (⑰).
11. Empty the remaining medication from the nebuliser and clean the device as described in the section «Cleaning and Disinfecting».
- When the liquid is getting low, you may tilt the device a little towards yourself in order to make full use of the residual liquid.
- The maximum tilting angle of the device is 45° while nebulising.

## EN

Do not shake the device during operation.  
This may cause the device to malfunction and/or shut down.

### 4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment.

Always wash your hands well before cleaning and disinfecting the accessories.

- Clean the main unit only with a soft, dry cloth.
- Do not expose the main unit to water or heat.

#### Cleaning with water

1. Snap the nebuliser lock (①-B) upward to open the nebulising head (③). Fill the drug storage area with hot tap water (max. 70 °C). Close the nebulising head and shake for 30 seconds.
  2. Run the device for 1-2 minutes in order to clean the nebulising head.
  3. **Wash all nebuliser components (except main unit) under warm tap water (max. 70 °C) for about 5 minutes** adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer.
  4. Rinse thoroughly making sure that all deposits are washed away and leave to dry.
  5. Store the device with all parts in a dry, clean place.
- Do not boil the main unit, nebulising head, mouthpiece and masks.

**Turbo cleaning function**  
We recommend using the Microlife mains adapter, as the power consumption of the turbo cleaning function is very high.

1. Snap the nebuliser lock (①-B) upward to open the nebulising head (③). Fill the drug storage area with hot tap water (max. 70 °C). Close the nebulising head and shake for 30 seconds.
2. Plug the adapter cable into the mains adapter socket (⑧) in the main unit (③).
3. Plug the adapter plug into the wall socket.
4. Press the ON/OFF button (⑥) for at least 5 seconds until the indicator light (④) starts blinking green.
5. Run the device until the liquid has been used up; the device will shut down automatically.

**Disinfecting by using disinfectants**  
All nebuliser components (except main unit) can be disinfected with chemical disinfectants following dosage and use limitations as provided by disinfectant manufacturer.

Disinfectants are usually available at pharmacies.

### 5. Battery Replacement

1. Open the battery compartment (⑦) on the bottom of the device.
2. Replace the batteries – ensure correct polarity as shown by the symbols in the compartment.

Batteries and electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

### 6. Maintenance, Care, and Service

We recommend replacing the nebulising head, face masks and mouthpiece every year (depending on the usage). Order all spare parts from your dealer or pharmacist, or contact Microlife-Service (see foreword).

### 7. Malfunctions and Actions to take

Use following table for solutions if any problem occur.

Description	Symptom/Possible causes	Solutions
Lower nebulising rate.	1. Indicator light lights up red. 2. Nebulising head is blocked.	1. Replace the batteries. 2. Clean the nebulising head - see section «Cleaning and Disinfecting»; replace it if necessary.
No power and/or nebulisation.	1. Batteries polarity are reversed. 2. Low battery voltage. 3. Adapter is not correctly connected to the main unit or the power socket. 4. Incorrect installation of nebulising head.	1. Reinstall batteries correctly. 2. Replace the batteries. 3. Reconnect the cable to the power socket and the main unit. 4. Reinstall the nebulising head correctly. See section «Preparation before Use».
Indicator light does not work.	1. Incorrect installation of nebulising head. 2. Contamination to the nebulising head. 3. The liquid is not in contact with the nebulising head. 4. There are bubbles between the liquid and the nebulising head. 5. Dirty electrode pole of the main unit.	1. Reinstall the nebulising head correctly. 2. Clean the nebulising head; replace it if necessary. 3. Slightly tilt the main unit towards yourself to make the liquid gather at the nebulising head side. 4. Gently shake the liquid. 5. Clean the electrodes of the main unit with a soft, dry cloth.

Contact your local Microlife-Customer Service if any problems remain after taking the measures listed above.

### 8. Guarantee

This device is covered by a **2 year guarantee** from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the receipt.

The nebulising head is covered by a 12 months guarantee and the replaceable parts like mouthpiece and masks are not included.

Opening or altering the device invalidates the guarantee.

The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.

### 9. Technical Specifications

<b>Nebulisation rate:</b>	0.2 ml/min.
<b>Particle size:</b>	≈ 5 μm (MMAD)
<b>Ultrasonic frequency:</b>	103 – 123 kHz ± 1k
<b>Acoustic noise level:</b>	< 30 dBA
<b>Nebuliser capacity:</b>	min. 2 ml; max. 8 ml
<b>Operating conditions:</b>	10 - 40 °C / 50 - 104 °F 30 - 80 % relative maximum humidity
<b>Storage conditions:</b>	-20 - +70 °C / -4 - 158 °F 10 - 85 % relative maximum humidity
<b>Weight:</b>	130 g (w/o batteries)
<b>Dimensions:</b>	121 x 44 x 59 mm
<b>Automatic switch-off:</b>	Automatically shut down, when the liquid has been used up.
<b>Voltage source:</b>	2 x 1.5 V alkaline batteries; size AA Mains adapter DC 3V, 1.33A (optional)
<b>Battery lifetime:</b>	approx. 90 minutes (using new batteries)
<b>IP Class:</b>	IP22
<b>Reference to standards:</b>	EN 13544-1; EN 60601-1; EN 60601-1-2; ISO 10933-5/-10; CE0120
<b>Expected service life:</b>	Device: 2 years Nebulising head: 12 months, when used 3 times/day for 15-20 minutes each time

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